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ALUMINIUM CONTAINERS WITH 1 VALVE

The aluminium container with one valve offers a system which allows to carry out a mechanical filtering. This system occurs in different phases which stand out for the placing of the valve.

The valve is in state of rest, before sterilization. During the vacuum phase, the valve opens completely, so that the air escapes from the container. Then, the valve closes and the counter valve opens; in this phase steam enters and the sterilization phase begins. At the end of the cycle the valve closes airtight and this will maintain the load sterile inside the container.

GENERAL FEATURES

- Completely made of aluminium alloy 5005, thickness 2 mm. Body and lid completely pressed without welding. Very sturdy.
- Gasket which ensures a perfect watertight, made of non-toxic silicone sponge that guarantees the maximum seal. Lock with locking system against casual opening. Seal application system.
- Label-holder for storage and classification.
- Handles for transport.
- Guaranteed anodizing for natural colour or other colours, thickness 20 microns.
- Lid colours: grey, yellow, blue, red, green.
- Stacking system compatible also with other manufacturing houses.

FOR AN EXCELLENT OPERATION, WE SUGGEST YOU TO ALWAYS MAKE THE FOLLOWING CHECKS BEFORE USING THE CONTAINERS:

- **Lid gasket:** check that the gasket is sufficiently elastic and that there are no cuts on it.
- **Locks:** check that the locks levers (hinges) sufficiently oppose the force supplied in the opening and that the pressure caused by the clamp is adequate to compress the silicon gasket.
- **Valve:** check the good functioning of the valve. It is necessary to open the lid and depress with a soft pointed rod the internal part of the valve, passing through the hole situated in the centre of the perforated aluminium plated.

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