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ALUMINIUM CONTAINERS WITH 2 FILTERS OR WITH 1 FILTER

If we compare stainless steel and aluminium alloy, we notice that the heat conduction coefficient of aluminium is 14,3 times higher; this means that, conditions being equal, the heat is transmitted 14,3 times more quickly through aluminium than through a similar piece of stainless steel. Moreover, thanks to the low specific density of aluminium alloy, it is possible to produce containers with a bigger thickness and with a slightly inferior weight.

This specificity has a great importance as to the heat conductivity and the sturdiness of the container.

Furthermore, thanks to the above mentioned features, it is well-known that at the end of the sterilization inside aluminium containers some excellent results are noticed about the drying of the content, while on the contrary in a stainless steel container we always notice an elevated degree of damp and the presence of condensation. That being stated, thanks to the collaboration had and researched by the operators of the sector, Titanox s.r.l. is pleased to introduce its new line of containers in aluminium alloy.

GENERAL FEATURES

- Completely made of aluminium alloy 5005, thickness 2 mm. Body and lid completely pressed without welding. Very sturdy.
- Gasket which ensures a perfect watertight, made of non-toxic silicone sponge that guarantees the maximum seal. Easily replaceable as well as the filters for steam passage.
- Lock with locking system against casual opening. Seal application system.
- Label-holder for storage and classification.
- Handles for transport.
- Guaranteed anodizing for natural colour or other colours, thickness 20 microns.
- Lid colours: grey, yellow, blue, red, green.
- Stacking system compatible also with other manufacturing houses.

FOR AN EXCELLENT OPERATION, WE SUGGEST YOU TO ALWAYS MAKE THE FOLLOWING CHECKS BEFORE USING THE CONTAINERS:

- **Lid gasket:** check that the gasket is sufficiently elastic and that there are no cuts on it.
- **Locks:** check that the locks levers (hinges) sufficiently oppose the force supplied in the opening and that the pressure caused by the clamp is adequate to compress the silicon gasket.
- **Filters:** check that the filters duration has not expired, that the filter is correctly positioned and that the filter is not cut or pierced.

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